# S250U HELPING HAND

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
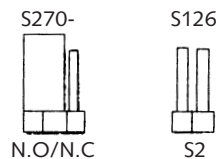
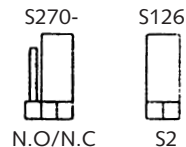
Kuva 7
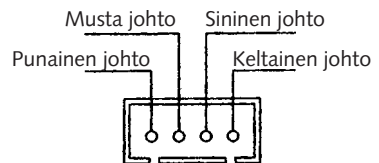
## MANUAL HELPING HAND S250U

Helping Hand includes a wireless transmitter (S250) and a wireless receiver (S270). The transmitter can either be hung around one's neck as a pendant, or be attached to a wall. This aid is especially useful for old and/or disabled people in need of help from outside.

- The transmitter sends out an emergency signal which is received by the receiver. The receiver will then emit a buzzing sound.
- The transmitter is battery-operated, while the receiver has a DC-in connector for a DC adapter.
- A 9V battery can be inserted in the receiver as backup.
- An optional external dialer can be connected to the receiver which is very useful during an emergency.

### Installation of Wireless Transmitter

#### Inserting a Battery

1. Use a Phillips screwdriver to loosen the screws on both sides of the transmitter, and open the case (see figure 1).
2. Insert a 2x1.5V battery type LR44 and make sure that the battery is correctly connected. (see figure 2).
3. LED indicator status: press the alarm release button at the bottom of the transmitter (see figure 3).

LED lights up: *Battery is good.*

LED lights up and flashes three times: *Battery is low.*

LED does not light up: *Battery is dead.*

#### Setting a House Code

Both the transmitter and receiver have a DIP switch with 8 switches. Each switch can be set to ON/OFF position.

The DIP switch settings of the transmitter and the receiver (also called the control unit) must match (see figure 4).

Figure 4

**CAUTION!** Do not set all switches to only either "ON" or "OFF" positions (see figure 5).

Figure 5

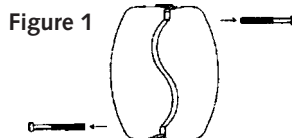
Figure 2



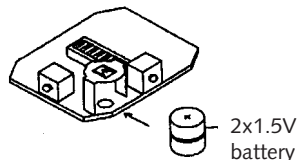
Figure 3

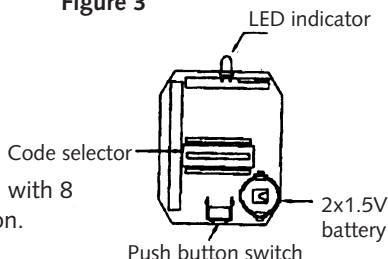After the DIP Switches are successfully set, use screws to close the case. This device should then be tested according to the instruction in the next section.

The transmitter can be worn around your neck, or it can be attached to the wall (see figure 6).

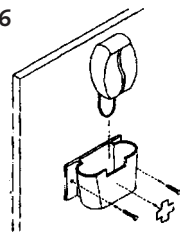
Figure 6

### Activating an emergency signal


It is advisable to wear the transmitter around your neck. In case of emergency hold the transmitter in your hand and press both halves together (see figure 7). The LED will light up. This indicates that a code is transmitted to the receiver.

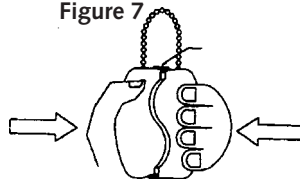Figure 7

If the transmitter is attached on the wall and an emergency occurs, pull down the transmitter's chain (see figure 8) (LED will light up) to send a code to the receiver. During an emergency, the transmitter will send an emergency signal 3 times every 5 seconds.

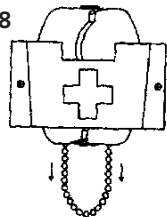Figure 8

### Deactivating an Emergency Signal

Use a ball point pen to press the rubber button at the bottom of the transmitter. The LED will then light up (see figure 9).

Figure 9

### Low Battery Indicator

Without pressing any button, the LED flashes continuously 3 times every minute.


While deactivating an emergency signal, the LED is lit for a while and then flashes three times.

## Installation of Wireless receiver

### Wall Attachment

The wireless receiver can be attached to a wall, or to a table. Use a screwdriver with a flat blade and carefully pry the case open. Attach the mounting plate at a suitable position (see figure 1).

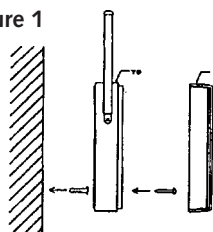Figure 1

### Power Supply

Connect the DC adapter to the DC-in connector on the left side of the receiver. Open the case as described above and insert a 9V battery as backup (see figure 2).

### Setting of House Code

Both the transmitter and receiver have a DIP switch with 8 switches. Each switch can be set to ON/OFF position.

Make sure that the DIP switches of both devices, transmitter and receiver, are set to identical positions, otherwise the transmitted signal cannot be received by the transmitter and the alarm will not be triggered (see figure 3).

**CAUTION! Do not set all switches to only either "ON" or "OFF" positions.**

The wireless receiver receives an emergency signal from the wireless transmitter, upon which it emits a buzzing sound.

The receiver contains a 9V battery as backup in case of a power failure.

### LED indicator status:

LED lights up: *DC adapter is connected and battery is good.*

LED lights up and flashes three times: *DC adapter is disconnected or the battery is low.*

LED does not light up: *DC adapter is disconnected, or the battery is dead.*

**CAUTION! After each alarm the device resets to normal operation after 1 minute.**

### Connecting an External Dialer

An external dialer (S126) can be purchased as a complement to the standard equipment. When old and/or disabled people stay at home alone and an emergency occurs, this external dialer connected to the wireless receiver will automatically dial a specific number for help from outside. The external dialer must be connected as shown in figure 4.

Figure 2

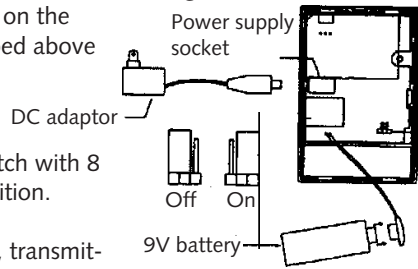
Figure 3


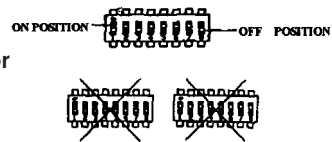
Figure 4

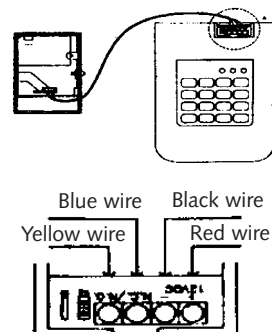
**CAUTION!** In order to connect an external dialer, the correct position of the jumper S1 must be set. Follow these steps to connect the external dialer:

If the jumper S1 is set to position N.O, the jumper S2 of the external dialer must be open (see figure 5).


If the jumper S1 is set to position N.C, the jumper S2 of the external dialer must be closed (figure 6).

In order to connect external accessories to the connector S3 of the wireless receiver, proceed according to the wiring diagram as shown in figure 7.


**Figure 5**

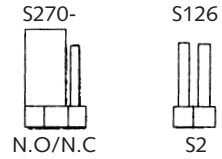**Figure 6**

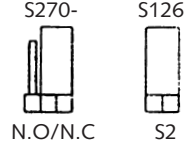**Figure 7**
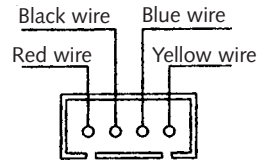
## Troubleshooting

- **The transmitter cannot trigger any alarm in the receiver.**

### 1. LED indicator does not light up:

Battery is low, or not properly connected.

*Check the battery's connection and condition.*

*Replace the battery if necessary.*

The connecting wires are loose.

*Check the connecting wires and reconnect them.*

### 2. LED indicator flashes

Battery is low.

*Replace the battery.*

DC adapter is disconnected.

*Reconnect the DC adapter.*

### 3. LED indicator is lit constantly.

The house code is set incorrectly.

*Match the house codes of the transmitter and receiver.*

The connector S3 has a short circuit.

*Change the wires of the connector S3.*

- **The LED of the transmitter does not light during a code transmission.**

The battery is low or not properly connected.

*Check the battery's connection and condition.*

*Replace the battery, if necessary.*

- **The LED of the transmitter is lit during a transmission, but the receiver does not emit any buzzing sound.**

The house code is set incorrectly.

*Match the house codes of the transmitter and receiver and test operation within 10 metres.*

- **The LED of the transmitter is also lit when there is no transmission.**

Battery is low, or not properly connected.

*Check the battery's connection and condition. Replace the battery if necessary.*

# **MALMBERGS**

Malmbergs Elektriska AB  
Box 144, 692 23 Kumla

Telefon: 019 - 58 77 00 Fax: 019 - 57 11 77  
[info@malmbergs.com](mailto:info@malmbergs.com) [www.malmbergs.com](http://www.malmbergs.com)